40 mm Screw Cap to ENPlus Adapter

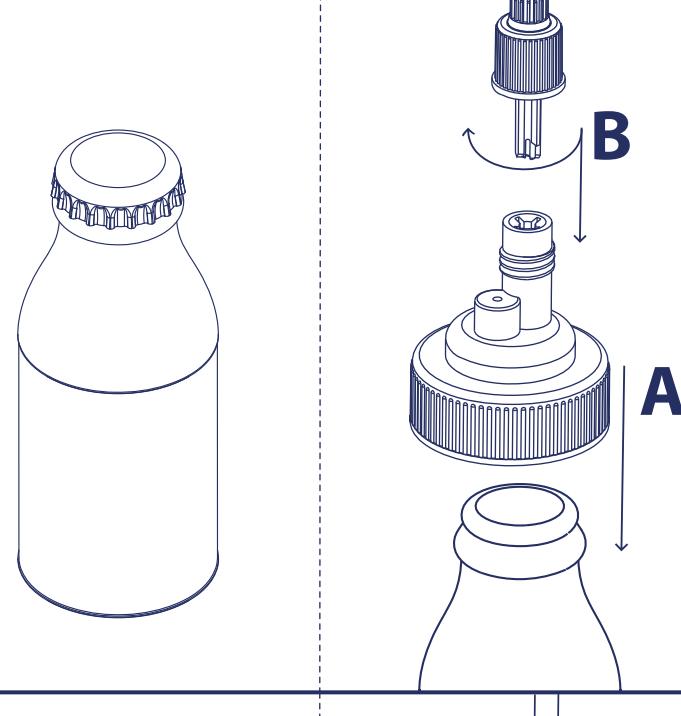
REF A120

0123

1

CS

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ΕΝ

Indications for use: This adapter is intended to connect enteral giving sets, equipped with ENPlus connector, to enteral nutrition crown cap bottles (Fig. 1) or wide neck reservoirs (Fig. 2) in order to deliver nutritional fluids. **Specifications:** Ready to use. Not made with natural rubber latex. Not formulated with DEHP or BPA. **Warnings:** Use ONLY with enteral feeding systems. NOT for intravenous use. This adapter does not protect against inadvertent misconnection with connectors of non-enteral healthcare applications. Do NOT use if package is open or damaged. Do NOT use if labeling is incomplete or illegible. **Duration of use:** Do NOT use for more than 24 hours. **Please note:** Do not reuse, as the connector may have been contaminated during use and become a source of infection. Follow good clinical practice and national/local guidelines for safe disposal. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision. **Instructions for use:** 1. Wash hands and use clean technique. 2. Remove protective cap (if applicable) from the enteral nutrition crown cap bottle/wide neck reservoir. 4. Securely snap the adapter into the enteral feed crown cap bottle (Fig. 1 - A), or securely screw the adapter onto the enteral feed wide neck reservoir (Fig. 2 - A). 5. Remove protective cap (if applicable) from the enteral giving set ENPlus connector. 6. Confirm the adapter is being connected to an enteral giving set. Do NOT connect the adapter to non-enteral giving sets. 7. Securely fit and screw (Fig. 1 - B, Fig. 2 - B) the ENPlus connector to the ENPlus port of the adapter. 8. Follow the enteral giving set instructions for use. 9. Discard the adapter together with the enteral giving set when delivery is complete within a maximum of 24 hours. Do not disconnect the adapter from the giving set.

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