
MIC-KEY* CONTINUOUS FEED EXTENSION SET WITH ENFit® CONNECTORS

SECUR-LOK® Right Angle Connector 2 Port "Y" ENFit® Connectors and Clamp

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Single Patient Use Only.

⚠ Warning: For enteral nutrition
and/or medication (Rx) only.

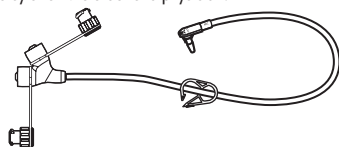	Nonsterile	Not made with natural rubber latex	Product is NOT made with DEHP as a plasticizer	Rx Only
	Caution		Consult instructions for use	REF 0141-12 0141-24

Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA.

In USA, 1-844-4AVANOS (1-844-428-2667). www.avanos.com

Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.

Avanos Medical Belgium BVBA, Leonardo Da Vincilaan 1, 1930 Zaventem, Belgium.

Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067 Australia.

*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.

© 2018 AVNS. All rights reserved. 2023-09-11

15-M1-333-03 / 50001870

www.mic-key.com

(EN) MIC-KEY® Continuous Feed Extension Set with ENFit® Connectors with SECUR-LOK® Right Angle Connector and 2 Port "Y" ENFit® Connectors and Clamp • Not made with natural rubber latex. • Product is not made with DEHP as a plasticizer. • Rx Only: Federal law (USA) restricts this device to sale by or on the order of a physician. • Single patient use only. • ⚠ Warning: For enteral nutrition and/or medication only. • The MIC-KEY® Extension Sets with ENFit® Connectors are intended to connect to the MIC-KEY® Low-Profile Gastrostomy Feeding Tube, Enteral Feeding Syringes with ENFit® Connectors, and Feed Administration Bags with ENFit® Connectors. • Device Aftercare: After each use, wash the Extension Set with warm, soapy water and rinse thoroughly. • Device Lifetime: It is intended to be used multiple times for a single patient for up to 14 days.

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